



One Hospital Drive • Lewisburg, PA 17837
Laboratory Services - (570)522-2510

ABO GROUP & RH TYPE

Test Code: BB

Specimen Type: Serum

Tube Type / Collection Container: BB (Brick color) or Red Top

Specimen Volume: 10 ML

Specimen Handling: Centrifuge and pipette serum into plastic screw cap transport tube.

Specimen Storage: Refrigerate 2-8 C

Specimen Collection Notes: Pipette serum from Red top labeled w/patient name, date, time of collection and collectors initials into a clean transport tube and label w/patient name, date, time of collection and collectors initials. Send Serum and Red Cells to the Laboratory. Call Blood Bank for details if transfusion is to follow. Serum stable 48 hrs refrigerated.

Department: Blood Bank

