



One Hospital Drive • Lewisburg, PA 17837  
Laboratory Services - (570)522-2510

## CPK MB FRACTION

**Test Code:** CKMB

**Specimen Type:** Serum

**Tube Type / Collection Container:** SST (Serum Separator Tube)

**Specimen Volume:** 1 ML

**Specimen Handling:** Centrifuge

**Specimen Storage:** Refrigerate 2-8 C

**Specimen Collection Notes:** Must draw discard tube prior to collecting a CPK specimen/Write draw off on appropriate tube and submit with CPK tube/Specimen stable 5 days refrigerated

**Department:** Chemistry

