



One Hospital Drive • Lewisburg, PA 17837  
Laboratory Services - (570)522-2510

## CERULOPLASMIN

**Test Code:** CERUL

**Specimen Type:** Serum

**Tube Type / Collection Container:** Brick Color Top, Red Top, Dk Green or SST (Serum Separator) **(Draw in Pre-Chilled tube)**

**Specimen Volume:** 1 ML

**Specimen Handling:** Centrifuge and pipette serum into plastic screw cap transport tube.

**Specimen Storage:** Refrigerate 2-8 C

**Specimen Collection Notes:** Must draw in Chilled Tube, keep on ice, Centrifuge immediately, separate serum into a transport tube, refrag immediately. Specimen stable 72hrs refrag- 4wks frozen

**Department:** Reference Laboratory

**Reference Lab Code:** 001560

