



One Hospital Drive • Lewisburg, PA 17837
Laboratory Services - (570)522-2510

FACTOR X ACTIVITY

Test Code: MISC

Specimen Type: Plasma

Tube Type / Collection Container: Light Blue; Buffered Sodium Citrate 3.2% **If Patient is known to have a HCT >55% Call ECH for**

Instructions

Specimen Volume: 2 ML Do NOT draw from an arm with a heparin lock or heparinized catheter

Specimen Handling: Centrifuge and pipette plasma into plastic screw cap transport tube.

Specimen Storage: Frozen

Specimen Collection Notes: Centrifuge, Separate plasma without disturbing cells at the bottom - Freeze. Call Lab for special collection instructions- Patient to avoid warfarin (Coumadin) therapy for 2 wks and heparin therapy for 2 days prior to test-Do Not stop taking med's without physician's approval-Do NOT draw from an arm with a heparin lock or heparinized catheter

Department: Reference Laboratory

Reference Lab Code: 086306

