



One Hospital Drive • Lewisburg, PA 17837
Laboratory Services - (570)522-2510

SCABIES PREP

Test Code: SCABI

Specimen Type:

Tube Type / Collection Container: Sterile Scalpel Blade


Specimen Volume:

Specimen Handling:

Specimen Storage: Ambient / Room Temp.

Specimen Collection Notes: Place a drop of mineral oil on a sterile scalpel blade. Allow the oil to flow onto a papule. Scrape vigorously six or seven times to remove the top of the papule. Transfer the oil and skin scrapings to a glass slide. Place a coverslip over the specimen on the slide. Transport to the laboratory in a cardboard slide folder.

Department: Microbiology

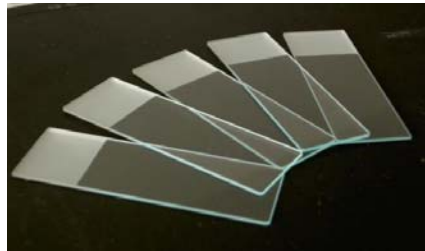

Transfer the oil and skin scrapings to a glass slide. Place a coverslip over the specimen on the slide.