



One Hospital Drive • Lewisburg, PA 17837
Laboratory Services - (570)522-2510

TOBRAMYCIN, RANDOM

Test Code: TOBRO

Specimen Type: Serum

Tube Type / Collection Container: Brick Color Top OR Red Top

Specimen Volume: 1 ML

Specimen Handling: Centrifuge and pipette serum into plastic screw cap transport tube.

Specimen Storage: Refrigerate 2-8 C

Specimen Collection Notes: Serum must be removed from cells within 1 hour of collection Pipette into plastic transport tube. Serum stable 2 hours room temp and 7 days refrigerated.

Department: Chemistry



OR

