



One Hospital Drive • Lewisburg, PA 17837  
Laboratory Services - (570)522-2510

## TOBRAMYCIN, TROUGH

**Test Code:** TTOBR

**Specimen Type:** Serum

**Tube Type / Collection Container:** Brick Color Top OR Red Top

**Specimen Volume:** 1 ML

**Specimen Handling:** Centrifuge and pipette serum into plastic screw cap transport tube.

**Specimen Storage:** Refrigerate 2-8 C

**Specimen Collection Notes:** Serum must be removed from cells within 1 hour of collection. Pipette into plastic transport tube. Serum stable 2 hours room temp and 7 days refrigerated. Collect immediately prior to dose, not more than 30 minutes prior.

**Department:** Chemistry



OR

