



One Hospital Drive • Lewisburg, PA 17837
Laboratory Services - (570)522-2510

TYPE AND SCREEN

Test Code: BB

Specimen Type: Serum

Tube Type / Collection Container: BB (Brick color) or Red Top

Specimen Volume: 10 ML

Specimen Handling: Centrifuge and pipette serum into plastic screw cap transport tube.

Specimen Storage: Refrigerate 2-8 C

Specimen Collection Notes: Green number I.D.system. Pipette serum from Red top labeled w/patient name, date, time of collection and collectors initials into a clean transport tube and label w/patient name, date, time of collection and collectors initials. Call Blood Bank for details; if transfusion is to follow. Serum stable 48 hrs refrigerated.

Department: Blood Bank

