



One Hospital Drive • Lewisburg, PA 17837  
Laboratory Services - (570)522-2510

## VALPROIC ACID

**Test Code:** VALP

**Specimen Type:** Serum

**Tube Type / Collection Container:** Brick Color Top OR Red Top

**Specimen Volume:** 1 ML

**Specimen Handling:** Centrifuge and pipette serum into plastic screw cap transport tube.

**Specimen Storage:** Refrigerate 2-8 C

**Specimen Collection Notes:** Serum must be removed from cells within 4 hours of collection. Specimen stable 48 hours refrigerated

**Department:** Chemistry



OR

