



One Hospital Drive • Lewisburg, PA 17837
Laboratory Services - (570)522-2510

VON WILLEBRAND MULTIMERS

Test Code: MISC

Specimen Type: Plasma

Tube Type / Collection Container: 2 Light Blue; Buffered Sodium Citrate 3.2%(x2)

Specimen Volume: 1 ML

Specimen Handling: Centrifuge and pipette plasma into plastic screw cap transport tube.

Specimen Storage: Frozen

Specimen Collection Notes: Call Lab for special collection instructions- Patient to avoid warfarin (Coumadin) therapy for 2 wks and heparin therapy for 2 days prior to test-Do Not stop taking med's without physician's approval-Do NOT draw from an arm with a heparin lock or heparinized catheter

Department: Reference Laboratory

Reference Lab Code: 500148

