



One Hospital Drive • Lewisburg, PA 17837
Laboratory Services - (570)522-2510

D-DIMER

Test Code: DDIMR

Specimen Type: Plasma

Tube Type / Collection Container: Light Blue; Buffered Sodium Citrate 3.2%

Specimen Volume: 2.7 ML Draw to fill line on tube-Short draws will be rejected.

Specimen Handling: Centrifuge at 3000rpm and pipette plasma into plastic screw cap transport tube ideally within 1 hr, not to exceed 4hrs.

Specimen Storage: Ambient / Room Temp.

Specimen Collection Notes: Stable 8 hours at room temperature.

Department: Chemistry

