



One Hospital Drive • Lewisburg, PA 17837
Laboratory Services - (570)522-2510

FOLIC ACID

Test Code: FOLI

Specimen Type: Serum

Tube Type / Collection Container: SST (Serum Separator) Wrap in Foil or Dk Colored Paper.

Specimen Volume: 2 ML

Specimen Handling: Centrifuge and Protect from Light immediately after collection. Stable 7 days. If also ordering a Ferritin and/or Vitamin B12 with Folic Acid, one Gold SST tube is sufficient. Ferritin Stable 5 days.

Specimen Storage: Refrigerate 2-8 C

Department: Chemistry

